



Venoplant **Trauma gel**

INDICATIONS

Venoplant trauma gel is a medical device that acts by protecting the wound that has injured the dermis in case of traumas, bruises, superficial wounds (abrasions and excoriations) and bumps. It creates an optimal environment for tissue repair processes while also protecting the injured area from external physical agents (rubbing) and bacteria. The mechanism by which the medical device acts is based on the ability to promote a film-forming effect by polymers and the polysaccharide component on the injured area, thus providing relief, also thanks to the moisturising action exerted by the polymeric component in the formulation

Composition: Isocide IMU, Centella asiatica dry extract, Melilotus dry extract, Cosmedia SP, Sodium hydroxide, Distilled water of witch hazel, Disodium EDTA, Glycacid ECO, Olive milk, Symocide PC, Phenoxyethanol, Eucalyptus, Natural camphor, water.

Posology and how to use it

The device is intended for non-invasive use (skin application).

Apply to the area to be treated. Gently massage in circles until product is completely absorbed.

Apply 2 to 3 times a day.

The treatment duration depends on the evolution of symptoms. The treatment should not last more than 30 days.

Presentation

Sale box: 40 g aluminium tube with Information leaflet

Warnings

- For topical use only.
- Keep out of the reach of children.
- In case of concomitant drug treatments, consult the doctor.
- No studies have been conducted in pregnancy and lactation. Therefore, in these cases, use the product only after consulting a doctor.
- Product functionality is guaranteed if the primary packaging is intact. Do not to use the device if the packaging is damaged or not fully intact when first opened.
- Do not use it after the expiry date specified on the package.
- Do not let it come into contact with the eyes. In case of accidental contact, thorough washing with warm water is recommended.
- Do not use in case of deep wounds, second and third degree burns
- Do not dispose the tube as household waste after use.

Interactions

No interactions related to the use of the Venoplant Trauma medical device are known to date.

Contraindications

Venoplast Trauma has no specific contraindications. Do not use the product in case of known hypersensitivity or incompatibility with even one single component of the formulation.

Side effects

To date, no side effects related to the correct use of the product have been detected. Discontinue use and consult a doctor in case of skin inflammation.

It is important to tell your doctor or pharmacist if you notice any side effects that are not described in this information leaflet.

Minimum storage life

24 months.










Validity after first opening: 3 months.

Storage

Store in a cool, dry place, away from heat sources, at a controlled temperature not above 25°C. The expiry date applies if product is properly stored and its package is not damaged. Do not dispose of the product or its package as household waste after use.

Net quantity: 40 g e

Rev. 01/2021

Symbols used on the package					
	Manufacturer		Do not use if package is damaged		Medical device in compliance with Directive 93/42/EEC as amended
	Batch code		Consult instructions for use, before use		Caution
	Use-by date		Non-sterile medical device		Store at temperatures not above 25°C



AESCULAPIUS
FARMACEUTICI

AESCULAPIUS farmaceutici s.r.l. - Via Cefalonia, 70 - Brescia

REF

912916307387